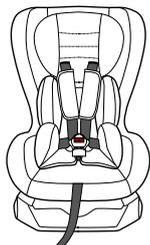
EN

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**CAR SEAT „CO-PILOT“  
INSTRUCTIONS FOR USE**

This child restraint complies with ECE regulations for use by children:

- ECE Group 0 / I
- User weight: Birth to 18 kg

Suitable for infants from birth to 10 kg (Rear-facing)  
Suitable for children from 9 kg to 18 kg (Forward-facing)

**READ THIS MANUAL**

Do not install or use this child restraint until you read and understand the instructions in this manual and in your vehicle owner's manual.

**FAILURE TO PROPERLY USE THIS CHILD RESTRAINT INCREASES THE RISK OF SERIOUS INJURY OR DEATH IN A SHARP TURN, SUDDEN STOP OR CRASH.**

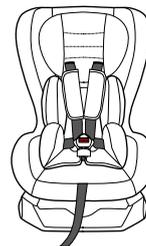
**IF YOU HAVE ANY PROBLEMS, PLEASE DO NOT RETURN THIS PRODUCT TO THE PLACE OF PURCHASE. PLEASE CONTACT CUSTOMER SERVICE AT +49 4761 886 0 OR AT [WWW.THOMASHILFEN.COM](http://WWW.THOMASHILFEN.COM)**

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CE

If you need help.....	3
Warnings .....	3
General warnings .....	3
Usage warnings .....	5
Injury warnings .....	6
Vehicle warnings .....	7
Certifications .....	7
Airbag warnings .....	8
Damage warnings .....	9
Mass and height limitations .....	10
Rear-facing .....	10
Forward-facing.....	11
Maximum height .....	11
Parts list .....	12
Vehicle seating positions .....	14
Safe vehicle seat belt systems .....	15
Unsafe vehicle seat belt systems .....	16
Preparing child restraint for use .....	17
Adjusting harness shoulder straps .....	17
Recline .....	19
Installing with lap-and-shoulder belt system .....	20
Installing rear-facing .....	20
Installing forward-facing .....	21
Using child restraint .....	23
Securing child in child restraint .....	23
Removing child from child restraint .....	25
Aircraft installation .....	25
Care and maintenance .....	25
Instructions for Use for Special Needs Version (Supplement) .....	26

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## IF YOU NEED HELP

Please fill in the information below for future reference. The model number and manufacturing date can be found on a label on the underside of the child restraint.

**Model Number:** \_\_\_\_\_

**Manufacture Date:** \_\_\_\_\_

**Date of Purchase:** \_\_\_\_\_

Our Customer Service Department is standing by to help you with any questions or concerns you may have regarding installation, use, care and/or parts. Please have your product's model number and manufacturing date available when contacting us.

## WARNINGS

### GENERAL WARNINGS

The safety of your child and the effectiveness of this child restraint depend on:

1. **Selecting a suitable location** for the child restraint in your vehicle.
2. **Properly routing the vehicle seat belt.**
3. **Properly securing both your child and the child restraint in the vehicle** using a seat belt designed to restrain both the child restraint and your child. Many seat belts are **NOT** safe to use with this child restraint. Please review the sections "Safe/Unsafe vehicle seat belt systems" carefully. If in doubt about installing this child restraint, consult the vehicle owner's manual or call the Thomashilfen Customer Service Department.

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# **WARNING!**

**FAILURE TO PROPERLY READ, FOLLOW AND USE THESE INSTRUCTIONS AND THE CHILD RESTRAINT LABELS INCREASES THE RISK OF SERIOUS INJURY OR DEATH IN A SHARP TURN, SUDDEN STOP OR CRASH.**

Your child's safety depends on the proper installation and use of this child restraint. Even if this child restraint seems easy to install and use, it is **VERY IMPORTANT** to read and understand these instructions and the vehicle owner's manual.

Follow all labels on this child restraint and in these written instructions. **DO NOT INSTALL OR USE THIS CHILD RESTRAINT UNTIL YOU READ AND UNDERSTAND THE INSTRUCTIONS IN THIS MANUAL AND IN YOUR VEHICLE OWNER'S MANUAL.**

**NO CHILD RESTRAINT CAN GUARANTEE PROTECTION WHEN THE VEHICLE IS SERIOUSLY IMPACTED.** However, when correctly installed, a child restraint will substantially improve the chances for survival and reduce the risk of serious injury in most crashes.

Carefully follow the instructions in this manual and in your vehicle owner's manual. If there is a discrepancy between the two, the vehicle owner's manual must take precedence with regards to seat belts and/or airbags.

- ! DO NOT** allow children to play with this child restraint.
- ! NEVER** leave children alone in a vehicle under any circumstance.
- ! It is recommended** that children are not left in their child restraint system unattended.
- ! Set a good example by ALWAYS** wearing your seat belt. Make it a firm rule that the vehicle does not move until everyone is buckled up. **MAKE NO EXCEPTIONS.**

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- ! It is not advised to loan or pass along a child restraint. **DO NOT USE A SECOND-HAND CHILD RESTRAINT** or a child restraint whose history you do not know.
- ! When disposing a child restraint system, ensure to dispose of the unit in a way others cannot reuse it (e.g. use a dark trash bag, do not sell to a thrift shop or in a garage sale, etc.).
- ! Store the child restraint in a safe place when not in use. **DO NOT** place heavy objects on top of the child restraint while in storage.
- ! **DO NOT** remove any stickers or warnings as they contain important information.

### USAGE WARNINGS

## **WARNING!**

- ! **ALWAYS ASSIST YOUR CHILD WHEN GETTING INTO AND OUT OF THE CHILD RESTRAINT.**
- ! **ENSURE THAT THE CHILD RESTRAINT IS LEVEL, STABLE AND DOES NOT ROCK SIDE-TO-SIDE. IF THE CHILD RESTRAINT IS NOT STABLE, TRY ANOTHER SEATING LOCATION.**
- ! **DO NOT ADD ANYTHING BEHIND OR UNDER THE CHILD RESTRAINT.**

According to accident statistics, **CHILDREN ARE SAFER WHEN PROPERLY RESTRAINED IN THE REAR SEATING POSITIONS THAN IN THE FRONT SEATING POSITION.**

Please study the section "Vehicle seating positions".

- ! **DO NOT** use any load bearing contact points other than those described in the instructions and marked in the child restraint.
- ! **DO NOT** allow others to install or use this child restraint unless they read and understand all the instructions and labels.

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! **DO NOT GIVE THIS CHILD RESTRAINT** to someone else without also giving them this Instructions für Use. Additional Instructions für Use can be downloaded directly from [www.thomashilfen.com](http://www.thomashilfen.com).

! To ensure proper child restraint use, keep and store this Instructions für Use and refer to it when necessary.

! **DO NOT LEAVE A CHILD RESTRAINT UNSECURED IN YOUR VEHICLE.** In a collision, sharp turn or sudden stop, an unsecured child restraint can injure other occupants. Remove it or make sure that it is securely attached with the vehicle seat belt, even when not in use.

## INJURY WARNINGS

! **INJURIES MAY OCCUR EVEN IF YOU ARE NOT IN A CRASH** if the child restraint is not properly installed or if your child is not properly secured in the child restraint.

! The rigid items and plastic parts of a child restraint shall be so located and installed that they are not liable, during everyday use of the vehicle, to become trapped by a movable seat or in a door of the vehicle.

! **DO NOT** leave folding vehicle seats unlatched. In the event of a sharp turn, sudden stop or crash, a loose seat back could prevent the child restraint from protecting the child as well as it should.

! It is recommended that any luggage or other objects liable to cause injuries in the event of a collision shall be properly secured. In the event of a sharp turn, sudden stop or crash, loose objects will keep moving, potentially causing serious injuries.

! The child restraint shall not be used without the cover. It **MUST** always be used with the fabric cover provided by the manufacturer.

! The seat cover should not be replaced with any other than the one recommended by the manufacturer,

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because the cover constitutes an integral part of the restraint performance.

**! PARTS OF THE CHILD RESTRAINT COULD BECOME HOT ENOUGH TO BURN THE CHILD IF LEFT IN THE SUN.** Contact with these parts can injure your child. **ALWAYS** touch the surface of any metal or plastic parts before putting your child in the child restraint. Cover the child restraint with a towel or blanket when the vehicle is parked in direct sunlight. **KEEP AWAY FROM SUNLIGHT, OTHERWISE IT MAY BE TOO HOT FOR THE CHILD'S SKIN.**

### VEHICLE WARNINGS

- ! Use ONLY** the vehicle's 3-point lap-and-shoulder belt system when restraining this child restraint to the vehicle. **DO NOT** use this child restraint with a 2-point vehicle belt (lap-belt only or shoulder-belt only). Your child would be seriously injured in the event of a crash.
- ! DO NOT** use this child restraint with unsafe vehicle seat belts listed in section "Unsafe vehicle seat belt systems" of this manual.
- ! DO NOT** use this child restraint if it can't be properly installed. It is possible that your vehicle has no suitable seating position for this child restraint due to seat belt type, airbag system and/or lack of location of a tether strap anchor. Read and follow these instructions, and your vehicle owner's manual carefully.

### CERTIFICATIONS

This is a "Universal" child restraint. It is approved to the requirements of the European Standard for Child Safety Seat Equipment under Regulation No. 44, 04 series of amendments, for general use in vehicles and it will fit most, but not all, car seats.

A correct fit is likely if the vehicle manufacturer has declared in the vehicle handbook that the vehicle is

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capable of accepting a “Universal” child restraint for this age group.

This child restraint has been classified as “Universal” under more stringent conditions than those which applied to earlier designs which do not carry this notice.

If in doubt, consult either the child restraint manufacturer or the retailer.

! Only suitable for use in the listed vehicles fitted with 3-point with retractor safety-belts, approved to ECE Regulation No. 16 or other equivalent standards.

This restraint is **NOT CERTIFIED** for aircraft use.

This child restraint must **NOT** be used in homes, boats, buses, motorcycles, motor homes, or any other vehicles that have non-forward-facing seats, missing seat belts, or whose seat belts do not meet the requirements of this child restraint.

## AIRBAG WARNINGS

! **NEVER PLACE THIS CHILD RESTRAINT IN A VEHICLE SEATING LOCATION THAT HAS AN ACTIVE FRONTAL AIRBAG.** An inflating airbag may cause serious injury or death. Refer to your vehicle owner’s manual for information about airbags and child restraint installation.

This child restraint may be used in a rear seating position equipped with side impact airbags providing they are:

- Roof mounted;
- “Curtain” type deployment;
- Will **NOT** interfere with, impact or in any way reposition the child restraint when inflated.

! If a vehicle’s front seat location must be used, **ALWAYS** disable the airbag. Failure to disable the airbag system can result in serious injury or death. Follow the vehicle owner’s manual for instructions on disabling the front airbag system.

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Please refer to your vehicle owner's manual for airbag type, and for appropriate use with child restraints.

## DAMAGE WARNINGS

- ! It is recommended that the device should be changed when it has been subject to violent stresses in an accident.
- ! **DO NOT USE A CHILD RESTRAINT THAT HAS BEEN IN AN ACCIDENT OF ANY KIND.** An accident can cause damage to the child restraint that you cannot see.
- ! **DO NOT** use this child restraint if it is damaged in any way or has missing parts.
- ! **NEVER** use a cut, frayed or damaged vehicle seat belt or harness.
- ! **DO NOT** tamper with or modify the child restraint, for any reason. Any unauthorised or additions to the device without the approval of the Type Approval Authority can cause serious injury or death in a sharp turn, sudden stop, or crash.
- ! **DO NOT** use anything to raise the child restraint off the vehicle seat. Untested modifications, such as adding extra padding, could affect the safety of the child restraint and result in serious injury or death in a crash.
- ! **DO NOT, EXCEPT AS DESCRIBED IN THESE INSTRUCTIONS, MODIFY YOUR CHILD RESTRAINT, DISASSEMBLE ANY PART OR CHANGE THE WAY THE VEHICLE SEAT BELTS ARE USED.** This includes any accessories or parts by other manufacturers as non-tested accessories can affect the safety of the child restraint.

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## ⚠ WARNING!

**THIS CHILD RESTRAINT HAS DIFFERENCES IN INSTALLATION AND USAGE DEPENDING ON THE WEIGHT AND HEIGHT OF YOUR CHILD. FAILURE TO USE THIS CHILD RESTRAINT WITHIN THESE GUIDELINES MAY INCREASE THE RISK OF SERIOUS INJURY OR DEATH.**

**! WARNING:** Verify your child's height and weight frequently. **DO NOT** guess before choosing the proper setup for your child.

It is recommended that you **ALWAYS** confirm country laws with local authorities. Laws vary from country to country regarding minimum/maximum child restraint requirements for age, mass and height.

### REAR-FACING


To use this child restraint, your child **MUST MEET ALL** of the following requirements:

#### • GROUP 0

##### Rear-facing:

between 0 - 10 kg  
(0 - 22 lb.)

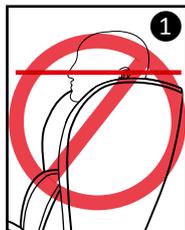- Child's ears **MUST BE BELOW** top of the headrest. If tops of ears are above the top of the headrest ①, the child is too large for the child restraint.

**! WARNING:** **DO NOT** use this child restraint forward-facing until the child has reached a minimum of 9 kg / 19.9 lb.

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

To use this child restraint, your child **MUST MEET ALL** of the following requirements:

• **GROUP I**

**Forward-facing:**

between 9 - 18 kg  
(19.9 - 39.7 lb.)

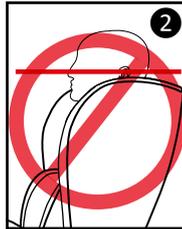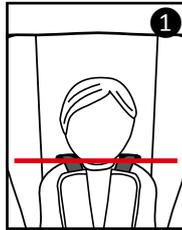
- Shoulders are **BELOW** top harness slots **1**.
- Child's ears **MUST BE BELOW** top of the headrest **2**. If tops of ears are above the top of the headrest, the child is too large for the child restraint.

MAXIMUM HEIGHT

**⚠ WARNING!**

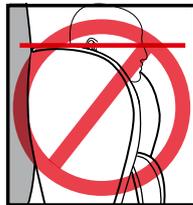Child is too tall when top of ears (mid-point of child's head) are above the top of the child restraint's headrest.

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- A. **Seat base**
- B. **Integrated belt-positioning guides**
- C. **Base belt guides  
(Forward-facing use only)**
- D. **Harness straps and buckle**
- E. **Shoulder pads**
- F. **Crotch strap and pad**
- G. **Metal up-front harness adjustment button  
(under fabric seat pad)**
- H. **Up-front harness adjustment strap**
- I. **Recline adjuster  
(red handle underneath front of seat)**
- J. **Head insert cushion (when applicable)**
- K. **Body insert cushion (when applicable)**
- L. **Harness splitter plate**

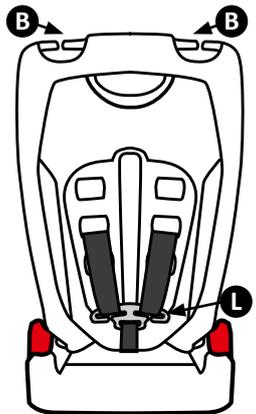
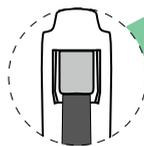
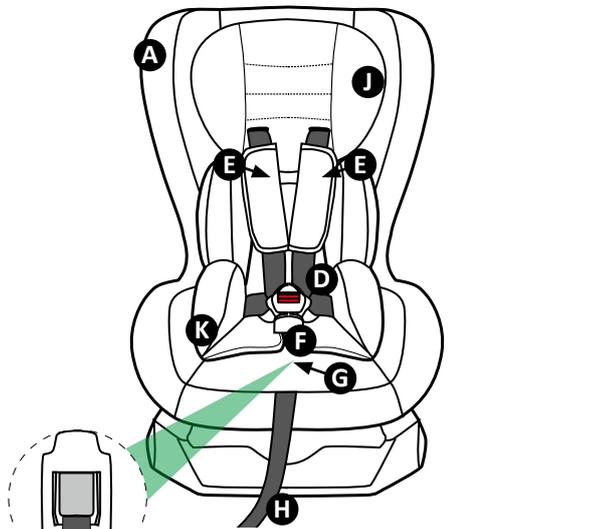
**TO PURCHASE REPLACEMENT PARTS:**

In Germany, please contact us at  
[www.thomashilfen.de](http://www.thomashilfen.de)  
 or by calling +49 4761 8860.

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## **WARNING!**

**FAILURE TO PROPERLY SECURE YOUR CHILD AND THE CHILD RESTRAINT IN YOUR VEHICLE INCREASES THE RISK OF SERIOUS INJURY OR DEATH IN THE EVENT OF A SHARP TURN, SUDDEN STOP OR CRASH.**

## **WARNING!**

**IMPROPER PLACEMENT OF THE CHILD RESTRAINT INCREASES THE RISK OF SERIOUS INJURY OR DEATH.** Some seating positions in your vehicle may **NOT** be safe for this child restraint. Some vehicles have **NO** seating positions which are compatible with this child restraint due to seating surface, seat belt type and/or airbag system. Read and follow these instructions, and your vehicle owner's manual carefully.

## **WARNING!**

Children aged 13 and under **MUST** ride in the rear seats, away from front airbags. The only exception is if the airbags are turned off. When using a child restraint in a vehicle with active airbags, refer to the vehicle owner's manual for additional installation instructions and precautions. For rear seat installation, adjust the front seat forward enough to ensure the child's feet do not hit the backrest of the front seat.

## **WARNING!**

Only use this child restraint in forward-facing vehicle seats. Side-facing or rear-facing seats in vans, station wagons or trucks **CANNOT** be used.

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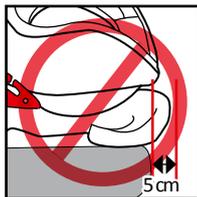
**THE VEHICLE SEAT MUST FACE FORWARD. DO NOT** use the child restraint with the following vehicle seats:

- Seats which face sides of vehicle.
- Seats which face rear of vehicle.
- Rear seats of small extended cab pickup trucks.

- The front of the child restraint **MUST NOT** hang over the edge of the vehicle seat by more than 5 cm (2 inches).

Refer to the vehicle owner's manual for child restraint installation instructions for your specific vehicle.

**! DO NOT** use this child restraint in a seating position where the vehicle seat belt or seat belt buckle is not easily accessible with the child restraint secured in place.

## SAFE VEHICLE SEAT BELT SYSTEMS

### **WARNING!**

**SEAT BELTS DIFFER FROM VEHICLE TO VEHICLE AND FROM SEATING LOCATION TO SEATING LOCATION WITHIN THE SAME VEHICLE. NOT ALL VEHICLE SEAT BELTS CAN BE USED WITH THIS CHILD RESTRAINT.**

You must determine if your vehicle's seat belts are compatible for use with this child restraint. If you are unsure, consult your vehicle owner's manual or call Thomashilfen.

Review your vehicle owner's manual for vehicle seat belt use with child restraints.

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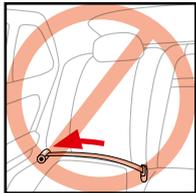
This child restraint can be installed properly using **MOST** vehicle 3-point lap-and-shoulder seat belts. However, some vehicle seat belts are not safe to use as their anchoring points for the lap belt may not allow for a secure installation. **NEVER** use any unsafe vehicle seat belt systems.


**DO NOT USE ANY OF THE FOLLOWING VEHICLE SEAT BELT SYSTEMS WITH THIS CHILD RESTRAINT:**

**! NEVER** use a lap-only belt.


**! NEVER** use a non-locking vehicle seat belt.

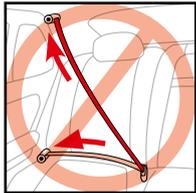**! NEVER** use motor driven automatic seat belts.


**! NEVER** use a vehicle seat belt system where the lap and shoulder belts have independent retractors.

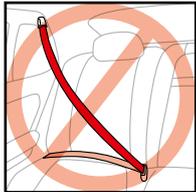**! NEVER** use a vehicle seat belt system with top or bottom anchorage points attached to the door in any way.

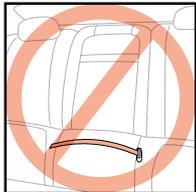**! NEVER** use lap belts whose anchor points are forward of the seat crease.

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## ⚠ WARNING!

**INCORRECT USE OF THE HARNESS SYSTEM IS THE MOST COMMON MISTAKE PEOPLE MAKE. IT IS VERY IMPORTANT TO READ, UNDERSTAND AND FOLLOW ALL INSTRUCTIONS AND WARNINGS. DO NOT MISTAKE COMFORT FOR SAFETY. THE HARNESS SYSTEM MUST BE SNUGLY ADJUSTED AT ALL TIMES.**

**! WARNING:** When used rear-facing, the shoulder harness must be **EVEN OR SLIGHTLY BELOW** the child's shoulders.

**! WARNING:** When used forward-facing, the shoulder harness must be **EVEN OR SLIGHTLY ABOVE** the child's shoulders. If the upper most harness position is below the child's shoulders, **DO NOT** use this child restraint.

**! ENSURE** the harness height is correct for your child's size. **VERIFY OFTEN.**

**! WARNING: DO NOT** use this child restraint if the midpoint of your child's head is above the top of the child restraint.

## ADJUSTING HARNESS SHOULDER STRAPS

1. Loosen harness system by pushing the up-front metal harness adjuster at the front of the seat just above the adjustment strap (under the fabric) ① and by simultaneously pulling the chest straps towards you ②.
2. To determine what harness height is appropriate for your child, place your child in the child restraint with their back flat against the backrest.

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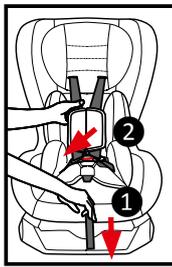
3. To adjust the height of the harness, remove both sides of the shoulder straps from the metal splitter plate on the rear of the seat (see "Removing/installing harness").

4. Gently pull the straps out from the front harness slots.

5. Select the proper harness slot height and feed the straps back through. **ENSURE THAT THE HARNESS STRAPS ARE THREADED AT THE SAME LEVEL, ARE NOT TWISTED AND ARE IN THE CORRECT SLOTS.**

6. Reattach the harness strap to the metal splitter plate ③.


**! ENSURE BOTH HARNESS STRAPS ARE PROPERLY SECURED ON THE SPLITTER PLATE.**

## **⚠ WARNING!**

**FAILURE TO REINSTALL THE HARNESS STRAPS CORRECTLY COULD RESULT IN SERIOUS INJURY OR DEATH.**

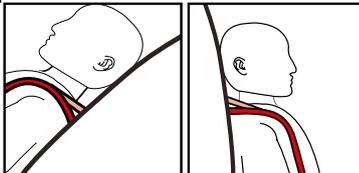**REMEMBER** to check the shoulder straps position frequently and move them to the next position when necessary. When **REAR-FACING**, the harness straps **MUST** be in the height position **EVEN** with or just **BELOW** the shoulders. When **FORWARD-FACING**, the harness straps **MUST** be in the height position **EVEN** with or just **ABOVE**

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the shoulders. The shoulder pads and crotch pads are optional and are not a safety feature.

## RECLINE

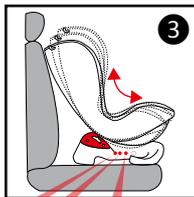
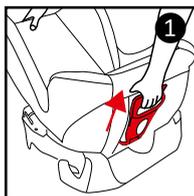
# ⚠ WARNING!

**YOUR CHILD RESTRAINT RECLINES TO ENSURE PROPER INSTALLATION INTO YOUR VEHICLE. THE CHILD RESTRAINT MUST BE INSTALLED SO THAT ALL OF ITS SURFACE AREA IS IN CONTACT WITH YOUR VEHICLE SEAT.**

Place the child restraint in the desired seating position (see “Choosing an appropriate vehicle seat location”). Pull the red recline handle found underneath the front of the seat and adjust the base to the desired angle. Release the red handle when done and gently push on the seat to ensure it is firmly engaged. An audible ‘click’ will ensure it is locked in place ①.

**WHEN INSTALLING THE SEAT REAR-FACING, THE SEAT MUST BE IN THE FULLY RECLINED POSITION (POSITION #1) INDICATED IN BLUE ②.**

**WHEN INSTALLING THE SEAT FORWARD-FACING, THE SEAT CAN BE ADJUSTED IN POSITIONS #2, #3, OR #4 INDICATED IN RED ③. PLEASE USE THE RECLINE THAT ALLOWS FOR THE BEST AND TIGHTEST INSTALLATION.**

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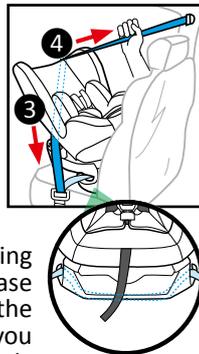
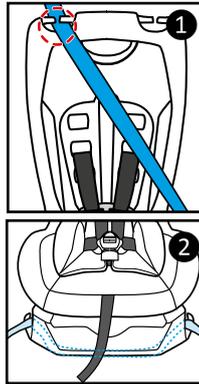
## INSTALLING REAR-FACING

**! MAKE SURE THE CHILD RESTRAINT IS IN THE MOST RECLINED POSITION #1 (SEE "RECLINE").**

1. Place the front of the child restraint flush against the back of a forward-facing vehicle seat.
2. Thread the shoulder belt portion of the vehicle 3-point seat belt through the integrated belt-positioning guide at the top of the headrest on the **SAME SIDE** as the seat belt retractor **1**.
3. Thread the lap portion of the vehicle 3-point seat belt over the base belt guides and **UNDER THE SEAT** as shown **2**.
4. Buckle the vehicle seat belt. Listen for an audible "click". The vehicle seat belt **MUST NOT** be twisted.
5. If your vehicle is equipped with a ELR/ALR switchable seat belt retractor system, we recommend that you pull out the full length of the vehicle seat belt (until you cannot pull out any more slack). This locks the retractor. If not equipped with switchable ELR/ALR retractor, continue with step #6.
6. Tighten vehicle seat belt by placing your knee on the child restraint base and use your full weight to push the child restraint seat down **3** as you simultaneously release the extra belt **4**.

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length manually back into the retractor until the seat belt is taught and has no slack ④.

For more information on how to tighten the seat belt for your seat belt system, see your vehicle owner's manual.

**! WARNING:** Rear-facing cars seats must be properly levelled:

- Too reclined can result in injury;
- Too upright can result in breathing difficulties.

**! ENSURE** that your infant is resting in the car seat in a way where their head cannot fall forward easily on its own.

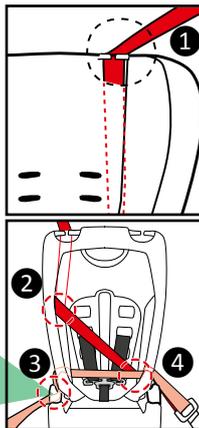
**! ENSURE** that your infant's bottom and back are flat against the child restraint and that your infant does **NOT** slouch.

## INSTALLING FORWARD-FACING

1. Place the child restraint flush against the back of a forward-facing vehicle seat. Adjust the recline as necessary (see "Recline").

2. Thread the vehicle shoulder belt portion of the seat belt through the integrated belt positioning guide at the top of the headrest **ON THE SAME SIDE** as the seat belt retractor ①.

3. Gently pull apart the hook and loop seam found on the backrest of the seat pad exposing the shell and EPE foam. Guide the shoulder portion of the vehicle 3-point seat belt along the backrest and completely through the opening on the side of the child restraint and out the back as shown ②.

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
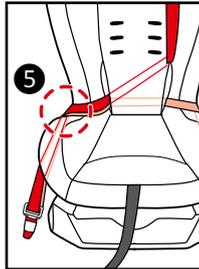
4. Thread the lap portion of the vehicle's seat belt through the red belt base positioning guide on the **SAME SIDE** as the seat belt retractor **3**.
5. From **BEHIND** the child restraint, pass both the lap and shoulder portions around the rear of the seat and back through the opening on the opposite side of the seat **4** as indicated.
6. Take both portions of the vehicle seat belt and guide them over the side of the child restraint **5**.

7. Buckle the vehicle seat belt. Listen for an audible "click". The vehicle seat belt **MUST NOT** be twisted.
8. Gently close the pad on the backrest which covers the shoulder portion of the vehicle seat belt.
9. If your vehicle is equipped with a ELR/ALR switchable seat belt retractor system, we recommend that you pull out the full length of the vehicle seat belt (until you cannot pull out any more slack). This locks the retractor. If not equipped with switchable ELR/ALR retractor, continue with step #10.

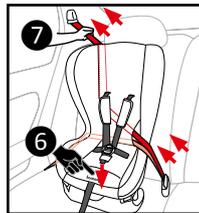10. Tighten vehicle seat belt by placing your knee on the child restraint base and by using your full weight to push the child restraint down **6** as you simultaneously release the extra belt length until seat belt is taught and has no slack **7**. Manually pull out of all of the slack from the **LAP PORTION** and then feed the shoulder portion back into the retractor.

For more information on how to tighten the seat belt for your seat belt system, see your vehicle owner's manual.

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
## ⚠ WARNING!

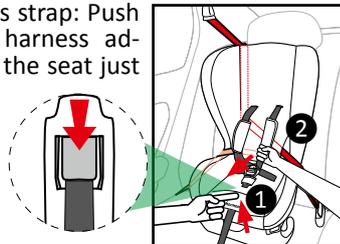**FAILURE TO FASTEN AND TIGHTEN THE HARNESS SYSTEM CORRECTLY MAY CAUSE THE CHILD TO BE EJECTED FROM THE RESTRAINT SYSTEM IN CASE OF A SHARP TURN, SUDDEN STOP OR CRASH CAUSING SERIOUS INJURY OR DEATH.** In a crash, the child restraint will provide **NO** protection for your child unless the harness straps are properly secured. **DO NOT** mistake comfort for safety. Harness system **MUST** be snugly adjusted. Follow these instructions carefully.

- ! Incorrect use of the harness system is the most common mistake people make. It is **VERY** important to read, understand and follow **ALL** instructions and warnings. **DO NOT** mistake comfort for safety. The harness system **MUST** be snugly adjusted at all times.
- ! Snugly adjust the harness provided with this child restraint around your child. A snug strap is one with no slack. It should not press on the child's flesh or cause your child's body to be in an unnatural position. You should only be able to slide **ONE FINGER** between the shoulder straps and the child's shoulder.
- ! Avoid unnecessary bulky clothing.

### SECURING CHILD IN CHILD RESTRAINT

1. To loosen the harness strap: Push the up-front metal harness adjuster at the front of the seat just **ABOVE** the adjustment strap (under the fabric) ① and simultaneously pull the chest strap towards you ②.

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
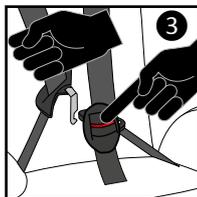
2. Unbuckle harness straps by pressing on the **RED** buckle release button and by pulling the buckle clips out **3**. Move harness straps to the side.

3. Place child in child restraint with child's back flat against back of the backrest.

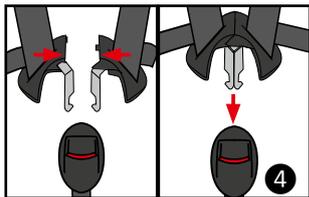4. Place harness straps over child's shoulders and insert the two-piece buckle clip into buckle until they 'click' into place **4**.


**! ENSURE** that the buckle is properly fastened by gently tugging on it.

**! ENSURE** that the straps are **NOT** twisted.

**! DO NOT** use child restraint if harness does not lock.

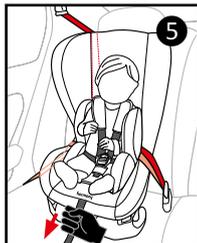5. Tighten harness system by gently pulling on the up-front harness adjuster strap until all slack is removed from the system **5**.

**! ENSURE** straps are flat and snug against child's shoulders and thighs. You should **NOT** be able to slide more than one finger between the strap and the child's shoulder **6**.

A snug strap should not allow any slack. It lies in a relatively straight line without sagging. It should not press the child's flesh or cause your child's body to be in an unnatural position.

**! Any** straps holding the restraint to the vehicle should be tight, any straps restraining the child should be adjusted to the child's body, and all straps should not be twisted.

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## REMOVING CHILD FROM CHILD RESTRAINT

1. Loosen the harness system by pressing on the up-front metal harness adjuster, and simultaneously pulling the shoulder straps towards you.
2. Unbuckle harness by pressing on the **RED** buckle release button.

**IF IN DOUBT ABOUT THESE POINTS OR ANY QUESTIONS RELATED TO INSTALLATION, PLEASE CONTACT THOMASHILFEN.**

## AIRCRAFT INSTALLATION

This child restraint is **NOT CERTIFIED** for aircraft use.

## CARE AND MAINTENANCE

### CLEANING THE PLASTIC SHELL:

- **SPONGE CLEAN** using warm water and mild soap.
- **TOWEL DRY.**
- **DO NOT** use solvents, abrasive cleaners, or disassemble child restraint in any way, except as instructed.

### CLEANING HARNESS STRAPS:

- Spot clean with a damp cloth. **DO NOT BLEACH.** **DO NOT** immerse the straps in water. Doing so may weaken the straps.

### CLEANING METAL AND PLASTIC HARNESS PARTS:

- Use mild soap and cool water on a damp cloth.

**! NEVER** lubricate the buckle. If buckle or harness adjuster sticks or will not remain tight, check for foreign objects such as food, dirt, etc. Remove objects with tweezers and/or clean with a cotton swab. If you cannot get the harness to remain tight with the buckle securely locked, **DO NOT USE** this child restraint. Call Thomashilfen to purchase a replacement harness or buckle.

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**INSTRUCTIONS FOR USE**  
**SPECIAL NEEDS CAR SEAT**  
**„CO-PILOT“**

Instructions for Use for Special Needs Version  
(Supplement)

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**ECE Gruppe 0/1**  
0-10 kg / 9-18 kg

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it. If you have any questions, please contact your authorised distributor or us directly.

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We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

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Preface / Manufacturer .....	27
Labels / Model Identification.....	29
General Remarks / Safety Instructions .....	30
Intended Purpose .....	32
– Intended Use / Application Areas .....	32
– Indication / Patient Target Group / Intended Users.....	32
– Risks Associated with Use and Contraindications .....	32
Product Description.....	33
– Scope of Delivery / Standard Equipment .....	33
– Options .....	33
– Putting into Service .....	33
– Teaching the User .....	34
Adjustment Options / Features .....	34
– Accessories .....	36
– Technical Data .....	37
– Material Specifications.....	37
Cleaning, Maintenance, and Repair .....	38
– Cleaning, Care Instructions, and Disinfection on a Daily Basis .....	38
– Maintenance and inspection by the distributor / Maintenance Schedule .....	39
– Spare Parts .....	41
– Storage / Disposal .....	41
EU Declaration of Conformity.....	42
Warranty.....	42
Period of Use / Service Life.....	42
Reuse .....	43
Distributor .....	43

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Label	Meaning
	<p><b>Barcode Label</b></p> <p>The bar code label is attached under the seat.</p> <ul style="list-style-type: none"> <li>- Producer</li> <li>- Product Name</li> <li>- Product Group</li> <li>- Item Code short / long</li> <li>- MD = Medical Device</li> <li>- Manufacture Date</li> <li>- Serial Number</li> </ul>
	<p><b>Type plate</b></p> <p>The type plate of the car seat is attached at the bottom of the seat.</p> <ul style="list-style-type: none"> <li>- Producer</li> <li>- Product Name</li> <li>- user weight</li> <li>- CE-Label</li> <li>- Follow Instructions for Use</li> </ul>

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## GENERAL REMARKS

**WARNING!**

Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from [www.thomashilfen.de](http://www.thomashilfen.de). The contents of the digital Instructions for Use can be enlarged.

Check the packaging for any transport damage and to be sure the product is in flawless condition.

Changes to the product are only permitted within the scope of the intended adjustment options.

The product can be cleaned daily and looked after by relatives and caregivers. Maintenance and inspection are done by the distributor.

In some countries it necessary to carry a medical certificate.

The rehab car seat can be used on the back seats as well as on the front passenger seat. The chosen car seat must face forward.

All labels have to remain on the product and should not be tampered with.

The Co-Pilot rehab child car seat has passed a load test in accordance with ECE R 44/04 requirements. ECE Group 0/1, 0-10 kg / 9-18 kg (0-22 lb. / 19.9-39.7 lb.).

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Check at least every three months that the settings on the product still work for the user's current body measurements. If need be, have a distributor / orthopaedic technician adjust the settings.

Do regular (weekly) checks to make sure that the setting options on the product work.

Be careful of the moving parts when adjusting or folding the product.

Please do not overload the product and note the maximum weight of the user and the maximum load capacity (see technical data).

Never leave the user unattended with the product.

The product should only be used by one user at a time. No other individuals should be placed in or on the product.

All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.

The user should always be positioned with the five-point positioning harness (standard equipment). The seat and the user always have to be fastened with the vehicle's own three-point seat belt.

Intense sunlight and heat can lead to high surface temperatures on the product. As a result, there is a risk of injury when touching the product. Only use the product once it has cooled down enough. In general, try to protect the product from direct sunlight.

Please note the safety standards which apply to your vehicle as outlined in the vehicle's Instructions for Use to ensure the safety of the user.

The Instructions for Use have to remain with the product.

If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

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## INTENDED PURPOSE

### INTENDED USE / APPLICATION AREAS

The Co-Pilot rehab car seat is a class 1 medical device.

It is designed to safely transport the physically disabled and to ensure children are sitting securely in vehicles. The seat is always fastened using the vehicle's own three-point seat belt. The five-point positioning harness in the rehab car seat helps to position and support the seating position of the child.

The rehab car seat provides additional support to help the passenger sit securely, stabilises head control, and offers secure seating for the hip area. The rehab car seat can be used as a rear-facing or a forward-facing seat.

Any use that goes above and beyond this does not comply with the intended use.

### INDICATION / PATIENT TARGET GROUP / INTENDED USERS

The rehab car seat is for children with structural and / or functional limitations affecting the head and torso or poor head and torso posture. Furthermore, it is suitable for children with limited body awareness and children of small stature. It provides a stable, physiological position if a standard child car seat does not securely support the child while seated in the vehicle.

### RISK ASSOCIATED WITH USE AND CONTRAINDICATIONS

When used appropriately, risks for the application can be excluded. Contraindications are not known.

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**PRODUCT DESCRIPTION****SCOPE OF DELIVERY**

Your product is delivered with the standard equipment and comes preassembled. Any accessories ordered along with the product are included and have to be assembled subsequently. A copy of the Instructions for Use is enclosed with the product.

**STANDARD EQUIPMENT**

Seat shell (rearward facing: 1 seating position / forward facing: 3 seating positions); seat cushion; head cushion; crotch pad; 5-point harness with central adjuster; abduction block; hip supports (coloured, matching the basic model); removable covers

**COLOR OPTIONS**

Co-Pilot, grey  
 Co-Pilot, red  
 Co-Pilot, blue  
 Co-Pilot, green  
 Co-Pilot, magenta

**PUTTING INTO SERVICE**

Carefully remove the transport protection and the packaging.

Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.

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**FARBVARIANTEN**

Co-Pilot, grau  
 Co-Pilot, rot  
 Co-Pilot, blau  
 Co-Pilot, grün  
 Co-Pilot, magenta

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The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Trained staff should install the seat in the vehicle with the user and adjust the seat to the measurements of the actual user.

The user has to be shown all of product's functions as well as how to install and remove it from the vehicle.

## ADJUSTMENT OPTIONS / FEATURES

### HEAD PAD

Attach the head pad to the upper part of the back of the seat cushion by using the hook-and-loop straps.

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- 1 Befestigung des Kopfpolsters

## ABDUCTION BLOCK – PADDED

To attach, pull the strap of the abduction block through the rear slot in the seat insert. Feed it back through the front slot and attach it to the push button.

- 2 a Belt end through opening
- 2 b Push button on abduction block

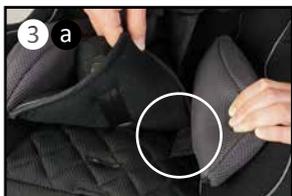
## HIP SUPPORTS

To attach the hip supports, you can simply stick them under the seat insert with the hook and loop strap.

The seat width can be increased by removing padding from the hip supports. To do this, open the padding with the zipper and remove the padding from the cover.

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### LATERAL TRUNK PADS, FOAM CORE (ACCESSORY)

Using hook-and-loop fasteners, the lateral trunk supports can be attached in various heights to the left and right in the back area.

④ Hook-and-loop fastening

### CAR SEAT PROTECTION MAT WITH BACK (ACCESSORY)

The car seat protection mat is under the child's car seat on the seat of the vehicle and the seatback. It protects the seat surface and the seatback of the vehicle from damage. Small items can be stored in the front pockets.

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## TECHNICAL DATA

Seat depth	29.5 cm / 11.6"
Seat width front	23.5 cm / 9.3"
(with side supports)	(20.5 cm / 8.1")
Seat width back	20 cm / 7.9"
(with side supports)	(14.5 cm / 5.7")
Back height seat	51 cm / 20.1"
Shoulder height belt guide	19/25/31 cm // 7.5/9.8/12.2"
Seat tilt-in-space (rear-facing)	45°
(forward-facing)	0° / 10° / 20°
Overall dimensions (W x H x D)	42 x 62.5 x 43 cm / 16.5 x 24.6 x 16.9"
Weight of seat	approx. 5 kg / 11 lbs
User weight	rear-facing: 0 - 10 kg / 0 - 22 lbs (group 0) forward-facing: 9 - 18 kg / 19.8 - 39.7 lb (group I)
Iso-Fix available	no

## MATERIAL SPECIFICATIONS

Seat shell: High density polyethylene (HDPE)

Padding: Polyester -- Trevira CS

The material used for the padding is "flame retardant" and was tested in accordance with DIN EN 1021-1 and DIN EN 1021-2.

The material and components underwent standard testing to determine the extent of their resistance to inflammation and were assessed as being very unlikely to catch fire.

The 100% Trevira CS material is a high-quality, breathable material and tested positive for biocompatibility. (DIN EN ISO 10993-5 and 10993-10 Trevira CS, black)

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CLEANING, CARE INSTRUCTIONS, AND  
DISINFECTION ON A DAILY BASIS**Padding**

The padding can be easily removed in a few simple steps and washed separately. The padding consists of a fabric cover and sometimes fillers made of foam. Check if the padding has a zipper or a hook and latch fastener. If it does, open the padding and remove the foam filler before washing the cover.

Please also note the following care instructions.

We would like to point out that even high-quality materials can fade as a result of long and intense exposure to sunlight or frequent washing.

**Hook and Loop Fasteners**

To keep the hook and loop fasteners working well, clean them now and then with a brush. Always close or cover the hook and loop fasteners before washing.

**Plastic Parts**

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

**Metal Parts**

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

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## Disinfection

Clean the padding, plastic parts, and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

## MAINTENANCE AND INSPECTION BY THE DISTRIBUTOR

### Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user's responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion.

If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

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# MAINTENANCE SCHEDULE – ANNUAL INSPECTION

	Area	Function	Damage	Replace
1	Plastic Body	👁	👁	
2	Adjustment Options	👁	👁	
3	Padding		👁	
4	Belts	👁	👁	
5	Plastic Parts	👁	👁	
6	Screws and Nuts	👁	👁	
7	Labels		👁	
<b>Accessory</b>				
8	Padding		👁	
9	Belts	👁	👁	
10	Adjustment Options	👁	👁	
11	Plastic Parts	👁	👁	
12	Metal Parts	👁	👁	
13	Screws and Nuts	👁	👁	
done by:			on:	
👁 = Inspection   ✓ = Test completed				

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Only original spare parts should be used for repairs. A list of the spare parts is available at [www.thomashilfen.de](http://www.thomashilfen.de). To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

### STORAGE

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

### DISPOSAL

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

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## EU DECLARATION OF CONFORMITY

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at [www.thomashilfen.de](http://www.thomashilfen.de) or requested from us directly.

## WARRANTY

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

## PERIOD OF USE

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

The period of use ends before the given time frame if the user has reached the maximum body weight or maximum body height to safely use the product.

The period of use does not apply to wear parts like padding, covers, etc.

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**SERVICE LIFE**

The product has a service life of 8 years if the requirements stipulated for the period of use are met. The service life consists of the period of use and storage times.

**REUSE**

The product is not suitable for reuse. The functionality of child car seats cannot be determined given a theoretical change of users. There could damage to the product that is not visible which rules out continued safe use. The sale and purchase of used child car seats is therefore ruled out by the manufacturer.

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Member of the International  
Support Association for the  
Rehabilitation of Children and Youths e.V.

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**Tendercare Ltd.**
Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstraße 1 · 27432 Bremervörde · Germany