



User Manual

*Excel  
Elise Travel Buggy*

*'Smart mobility solutions'*

Read instructions before use

# Table of Content

1. Feature
2. Warning
3. Measurement
4. Assemble
5. Function
6. Warranty and Maintenance

# 1. Feature

- A. Canopy
- B. 5-point harness and pad
- C. Basket
- D. Footrest
- E. Soft seat pad with batting
- F. Folding lock
- G. Brake



## 2. WARNING

READ AND FOLLOW ALL WARNINGS AND INSTRUCTIONS!  
FAILURE TO DO SO COULD RESULT IN SERIOUS INJURY  
OR DEATH!

- NEVER leave child unattended. ALWAYS keep child in view while in the pushchair.
- Avoid serious injury from falling or sliding out. ALWAYS use harness.
- When making adjustments to the pushchair, ALWAYS ensure that all parts of the child's body are clear of any moving parts of the pushchair.
  
- DO NOT use pushchair if any parts are missing or broken.
- DO NOT allow child to climb on or around pushchair.
- DO NOT allow child to stand on the footrest. pushchair may tip over and injure child.
- NEVER leave stroller on a hill or incline, even with brakes set, pushchair may tip injuring the child.
- To avoid accidental rolling, ALWAYS set brakes when pushchair is not being pushed.

### 3. Measurement

Specification	Size
Width (W)	55 cm
Height (H)	112 cm
Length (L)	96 cm
Footrest height (FL)	21,27,40 cm
Footrest dimensions	44 x 19 cm
Seat width (SW)	40 cm
Seat depth (SD)	27 cm
Back height	56 cm
Back width (BW)	40 cm
Product weight	6,9 kgs
Weight capacity	50 kgs
5-Point Harness (Restraint)	
Seat to Footrest Height Adjustment	



### 3.1 Accessory



Canopy



Lateral support



Ankle huger



Chest pad



Boot



Extra seat pad



Basket




Harness pad



Rain Cover

## 4. Assemble

### 4.1 Unfold pushchair

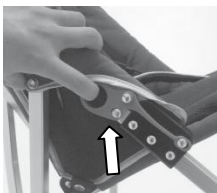

## 4.2 Canopy Assemble



## 4.3 Folding Pushchair



#### 4.4 Ankle huger assemble



#### 4.5 Chest and assemble






## 5. Function

### 5.1 Boot assemble



### 5.2 Fasten 5-point harness and chest pad

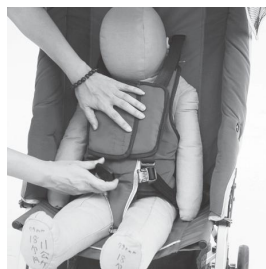

### 5.3 Brakes



Step on both the brakes to lock the pushchair.

### 5.4 Footrest height adjustment



## 6. Warranty and maintenance

There is a comprehensive 1 year warranty from the date on which your new pushchair is delivered. The warranty covers the stroller for repairs or replacement during this period.

For more detail, please see the warranty conditions:

1. Any work or replacement part installation must be carried out by an authorized dealer / service agent.
2. To apply the warranty should your pushchair require attention please contact the outlet from which you purchased the pushchair.
3. Should any part of the pushchair require repair or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. Note: The guarantee is not transferable.
4. Any repaired or replaced parts will be covered by the balance of the warranty period on the pushchair.
5. Parts replaced after the original warranty has expired will be covered by a three-month warranty.
6. Consumable items supplied will not generally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or material defect. Such items include (among others): upholstery and tyres.
7. The above warranty conditions apply to brand new pushchair.

8. Under normal circumstances, no responsibility will be accepted where the pushchair has required assistance as a direct result of:
- a. the pushchair part not having been maintained in accordance with the manufacturers recommendations
  - b. failure to use the manufacturer's specified parts
  - c. the pushchair or part having been damaged due to neglect, accident or improper use
  - d. the pushchair or part having been altered from the manufacturer's specification or repairs having been attempted before the dealer is notified

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data. All figures, measurements and capacities are approximate and do not constitute specifications.



For UK customers:

**Van Os Medical UK Ltd.**  
Excel House  
Ashbrooke Park  
Sherburn in Elmet  
North Yorkshire, LS25 6PJ

Tel.: +44 (0) 1977 681400  
Fax. +44 (0) 1904 720398  
E-mail: [sales@vanosmedical.com](mailto:sales@vanosmedical.com)  
[www.vanosmedical.com](http://www.vanosmedical.com)

For International customers:

**Van Os Medical B.V.**  
Koperslagerij 3  
4651 SK Steenberg (NB)  
Nederland

Tel. +31 (0) 167 57 30 20  
Fax. +31 (0) 167 57 33 81

E-mail: [info@vanosmedical.nl](mailto:info@vanosmedical.nl)  
[www.vanosmedical.nl](http://www.vanosmedical.nl)